

Arrow guide  Pull  Push  Point  Twist

Preparation

Before you start

Preparation 1

Before following the user steps described in this user guide, please make sure to carefully read the information and disclaimers in the bottom of the last page. Please note that the information provided in this guide is not a substitute for professional medical advice or for reading the instructions for use (IFU) that comes with the infusion set.



Preparation 2

Wash your hands before inserting the infusion set.



Preparation 3

Choose an insertion site as recommended by your healthcare provider. **Do not** use an area right next to your previous infusion site.



Preparation 4

Clean the area for insertion with a disinfectant as directed by your healthcare provider. Be sure the area is air dry before inserting the infusion set.

Insertion



Insertion 5

Open the blister pack and take out the contents.

Caution
Ensure sterility by checking if the packaging is undamaged. Do not use if the infusion set is already open or has been damaged.



Insertion 6

Pull gently to remove the paper from the adhesive tape. Be careful not to touch the adhesive tape.



Insertion 7

Remove the disconnect cover from the insertion device by gently squeezing the sides of the disconnect cover while pulling it straight out. Keep the disconnect cover for later use.



Insertion 8

Before placing the insertion device against the skin, note that the arrows on the insertion device show where the tubing will be connected after insertion.

Arrow guide



Pull



Push



Point



Twist

Insertion



Insertion **9**

Gently stretch the skin and ensure that the surface is flat and taut. Position the insertion device on the skin and press the release button completely down to insert the infusion set.



Insertion **10**

Gently remove the insertion device. The product is now in safe mode with the release button pressed down.



Insertion **11**


Gently massage the adhesive tape onto the skin. Change to a new product and a new site, if the adhesive tape becomes loose.



Insertion **12**


Remove the paper from the tubing. Follow your pump manufacturer's instructions for use to connect the infusion set tubing to the pump reservoir and pump.

Insertion

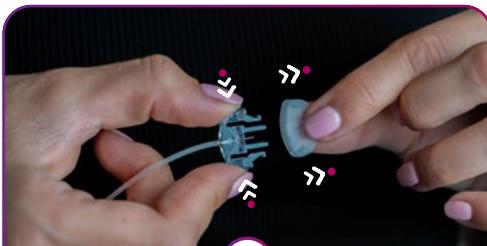

Insertion **13**

Remove the protective cap for the site connector by gently squeezing the sides of the site connector and then pulling the protective cap off. Keep the cap for later use.

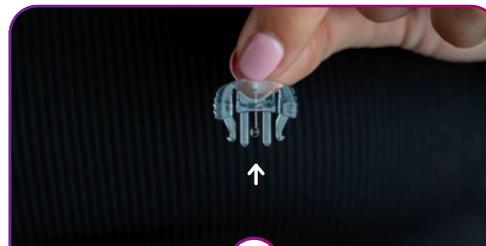

Insertion **14**

Hold the infusion set with the needle pointing down. Prime the infusion set according to the instructions provided by the pump manufacturer until insulin dispenses from the needle. Be sure there is no air in the tubing.


⚠ Caution
Never fill the tubing while the infusion set is connected to your body.



Insertion **15**

To connect the site connector to the cannula housing, place a finger on the cannula housing while pushing the site connector straight in until you hear a "click".

Frequently check the insertion site to ensure that the soft cannula remains in place and that the site is not irritated. Change to a new product and a new site if necessary. Remember to store the covers for later use.

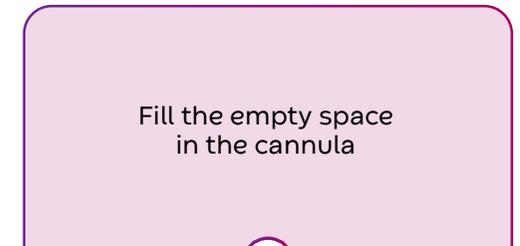

Insertion **16**

Fill the empty space in the cannula following your pump manufacturer's instructions.

⚠ Caution
Always check your blood glucose level one to three hours after insertion to ensure the infusion set is working properly. Measure your blood glucose on a regular basis. Discuss this with your healthcare provider.

Arrow guide



Pull



Push



Point



Twist

Disconnecting



Disconnecting

A

This infusion set allows you to temporarily disconnect your pump without having to change the infusion set. Place a finger on the cannula housing and gently squeeze the sides of the site connector. Pull the site connector straight out from the cannula housing.



Disconnecting

B

Place the protective cap on the site connector.



Disconnecting

C

Insert the disconnect cover into the cannula housing until you hear a "click".

Arrow guide



Pull



Push



Point



Twist


Reconnecting



Reconnecting

D


Remove the disconnect cover by placing a finger on the cannula housing and gently squeezing the sides of the cover. Pull the cover straight out of the cannula housing.

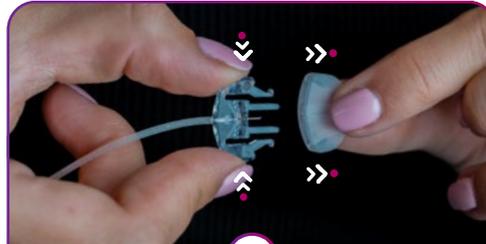

Reconnecting

E

Remove the protective cap from the site connector by gently squeezing the sides of the site connector and then pulling the protective cap off.

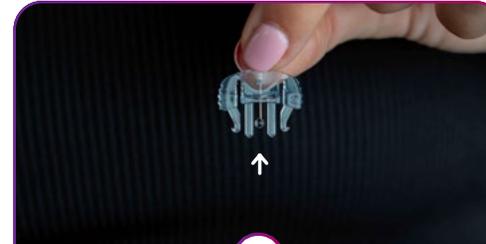

Reconnecting

F

Hold the infusion set with the needle pointing down. Prime the infusion set according to the pump manufacturer's instructions for use until insulin dispenses from the needle. Ensure there are no air bubbles in the tubing.

Caution

Never fill the tubing while the infusion set is connected to your body.



Reconnecting

G

To connect the site connector to the cannula housing, place a finger on the cannula housing while pushing the site connector straight in until you hear a "click".