

Application Card



How to Use

Convatec Little Ones® One-Piece Extra Small Stoma System*

Baseplate/Pouch Preparation and Application



1 **Measure** the stoma size using the measuring guide provided.



2 **Trace** the stoma outline you have selected onto the release paper on the baseplate. The small starter hole should be in the centre of your tracing.



3 **Important:** Before cutting, separate the front and back panels of the pouch to avoid cutting through the pouch. Pouch should contain a small amount of air. Cut along the tracing with curved, short-blade scissors.



4 Cut the centre hole the same size as the stoma, using guidelines on the release paper. Do not cut beyond the outer line.



5 Prior to applying the pouch, it is recommended that you close the InvisiClose® clipless tail closure system by folding upwards and pressing the interlocking closures together until you feel them lock into place. A "click" indicates a secure closure.

Attachment of Pouch



6 Ensure the skin around the stoma is clean and dry using warm water and soft wipes. Now remove the release paper from the baseplate.



7 Place the pouch gently in position over your child's stoma.



8 Gently press the baseplate against the skin, paying particular attention to the area closest to the stoma.



9 During application, maintain gentle pressure on the baseplate for 30 seconds to improve adhesion.

Pouch Removal



10 Once ready to remove the pouch, have a some water and some soft cloths ready to clean the skin.

To release the pouch from the skin, go slowly and gently.

Discard as necessary.

Please note: It is not necessary to use gloves, this is similar to changing a nappy. If you have been advised to use gloves by your clinician then continue to do so.

Application Card

InvisiClose®
Tail Closure System

How to Use InvisiClose® Tail Closure System*

These instructions apply to the Little Ones® Extra Small Stoma System

Directions to Close the Pouch




The InvisiClose® Tail Closure System is comprised of a **security flap (A)**, **interlocking closures (B)**, and **outlet end-strips (C)**.



Hold the **security flap** against the front of the pouch revealing the interlocking closures.



Fold the tail end up towards four times and ensure the interlocking closures overlap.

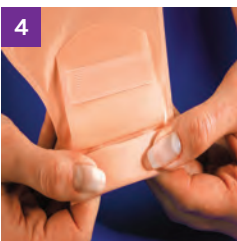

Press the interlocking closures together until you feel them lock (**click**) into place along their entire length.



Wrap the security flap around to the back of the pouch (**body side**) and press the interlocking closures together until you feel them lock (**click**) into place.

Tips:

- Remember to empty your child's pouch when it is one-third full.
- The use of soaps or cleansers in the pouch is not recommended. If desired, rinse only with tepid water.

Directions for Draining the Pouch



Turn the back of the pouch (body side) upwards. Carefully lift the security flap upwards to unlock.



While holding the security flap up against the front of the pouch, unlock the interlocking closure by lifting at the corner and unraveling the folds four times.



To facilitate draining and/or cleaning, you may open the tail end by pushing in both ends of the outlet end-strips with your thumb and forefinger. If desired, pouch tail may be cleaned with water. Pat dry.

For more information or if you have questions, call

0800 282 254

or visit our website at www.convatec.com/en-gb

