

# How to use the Stomahesive® Seal



### Getting Started - Assess the Skin:

- Make sure the skin is clean and dry.
- Warm the seal in your hands
- Remove the release liner from both sides.



### Customise to Meet Your Needs:

- Gently mould, roll, stretch or compress the seal to fit around the stoma. The seal may be cut, stacked or rejoined to fit around difficult areas.
- Gently apply and smooth the seal towards the base of your stoma. While applying a little pressure, hold the seal in place for about 30 seconds.



### Applying Your System:

- Place the Ostomy appliance directly on to the surface of the seal and hold in place for 30 seconds with gentle pressure until secure.
- Wear as long as it feels comfortable and secure.



### Removal:

- Gently peel the pouching system from the body. Any residue can be removed from the skin by rolling and peeling or by using ESENTA™ Sting Free Adhesive Remover.

**Note:** Do not use to cover broken skin or open wounds, or if you have developed a sensitivity or allergic reaction to this product.

See package insert for complete instructions for use.

## How to use the Stomahesive® Seal

### TIPS AND TRICKS

- ✓ The seal is supposed to swell. It gels when in contact with moisture to form a better seal and provide added skin protection from effluent.
- ✓ The seal can help with convexity. Use a seal under a convex skin barrier, or stack several seals to fill in uneven pouching surface.
- ✓ You can touch all sides of the seal, roll it, mould it - and it will not damage the adhesive.
- ✓ You can combine the Stomahesive® Seal and Stomahesive® Paste to meet your needs.



### Convatec Stomahesive® Seal

PRODUCT CODE	DESCRIPTION	QUANTITY	SAS CODE	MONTHLY ALLOWANCE
413504	48mm Thin	10	80008Y	30
413503	48mm Standard	10	80008Y	30
413505	98mm Large	10	80008Y	30

\* SAS Code for Australia Only



Find support on every step of your ostomy journey with the me+® program. To find out more or to request samples, please contact our customer care team:

Australia Call **1800 335 276** or email [connection.au@convatec.com](mailto:connection.au@convatec.com)  
New Zealand Call **0800 225 4309** or email [connection.nz@convatec.com](mailto:connection.nz@convatec.com)

See package insert for complete Instructions for Use.

ALWAYS FOLLOW THE DIRECTIONS FOR USE

®/™ indicates trademark of Convatec Inc. ©2024 Convatec Inc.  
AP-013770-MM O408.2 September 2024

